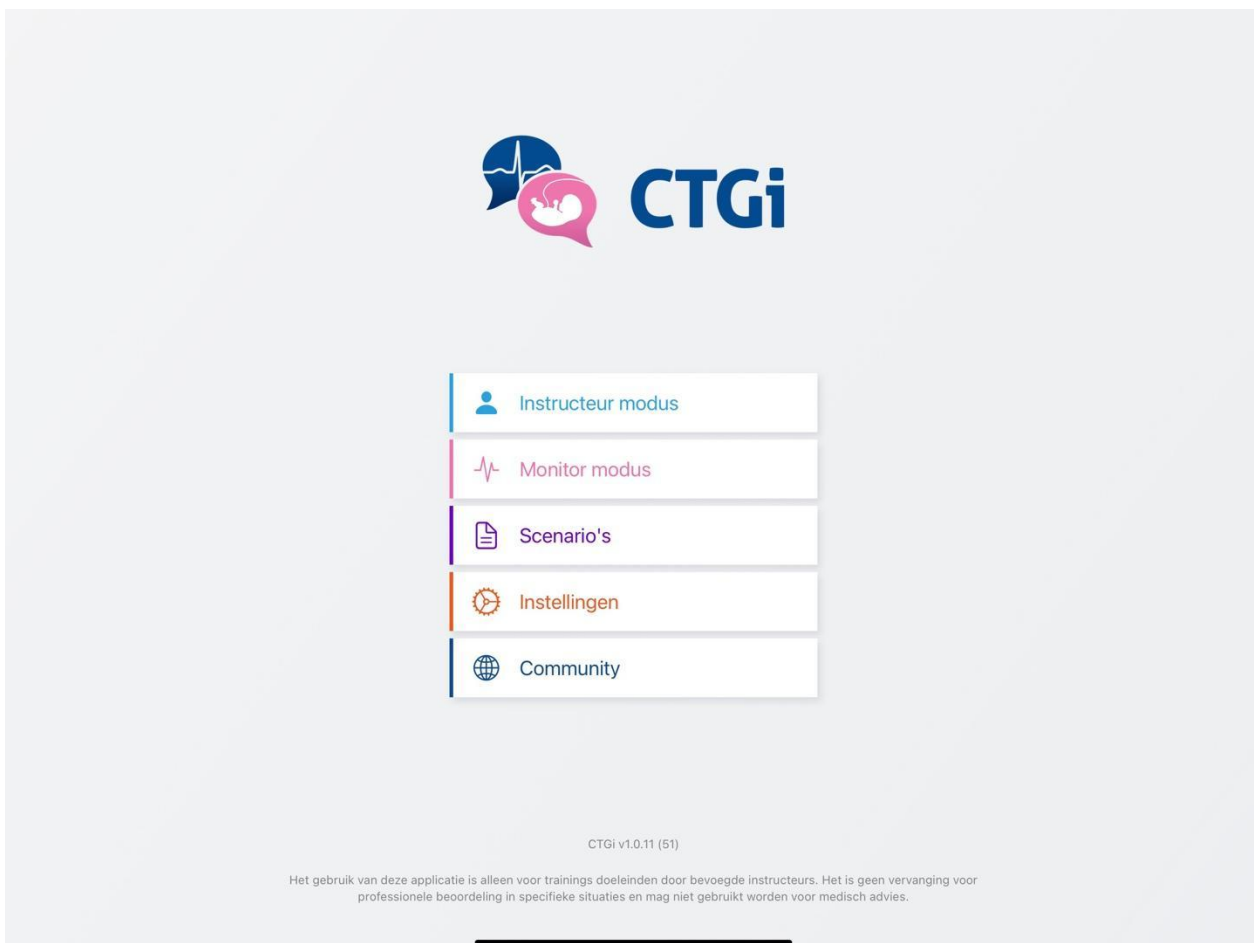
# CTGi – Set-up Gids

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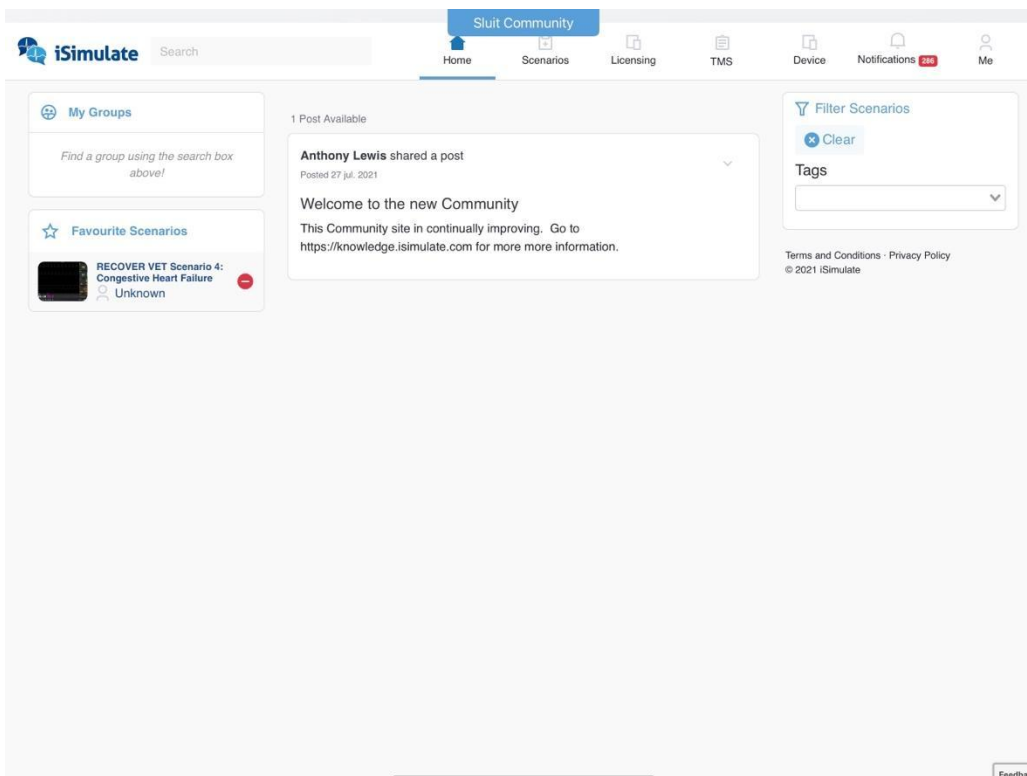
6. Klik op "Relinquish".

The screenshot shows the iSimulate user interface. At the top, there is a navigation bar with 'Close Community' and icons for Home, Settings, Learning, Tools, Device, Notifications, and Me. The left sidebar contains sections for 'My Profile', 'My Content', 'Provisioning', 'My Portfolio', 'Resources', and 'Support'. The main content area is titled 'Provision Device' and includes a 'Current License' card with a 'Relinquish' button. Below this are sections for 'Your Standalone Licenses' and 'Your License Agreements'. A blue banner states 'Your device is currently licensed to the agreement'. The 'Ctgi License Agreement' card shows 'You have 0 licenses available on this agreement' and a 'Relinquish' button. The details for the agreement are as follows:

Details	
Type	Recuring
License on agreement	1
License used	1
Agreement ID	2476

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The screenshot shows the iSimulate user interface. At the top, there is a navigation bar with the iSimulate logo, a search bar, and a 'Close Community' button. Below this are navigation tabs for Home, Simulators, Licensing (which is active), TME, Device, Notifications (with a red badge), and My. A left sidebar contains sections for 'My Profile' (My Profile, Update details, Change password), 'My Content' (My media, My scenarios, My groups), 'Provisioning' (Provision device), 'Pre-Release' (View pre-releases), 'Resources' (Available downloads), and 'Support' (Support information).

The main content area is titled 'Provision Device' and contains sub-sections for 'Your Standalone Licenses' and 'Your License Agreements'. A prominent card titled 'Ctgi License Agreement' is displayed, featuring a 'Claim' button. Below the card title, it states 'You have 1 licenses available on this agreement'. A 'Details' section lists the following information:

Type:	Recurring
Licenses on agreement:	1
Licenses used:	0
Agreement ID:	2470
Agreement Date:	2021-02-20

Below the details, a 'Modules Included' section contains a 'Ctgi Device' button. At the bottom of the page, there are links for 'Terms and Conditions' and 'Privacy Policy', and a copyright notice for © 2021 iSimulate.

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The screenshot displays the 'Details' page of a mobile management application. On the left is a navigation sidebar with sections: 'My Content' (My media, My scenarios, My groups), 'Provisioning' (Provision device), and 'Support' (Support information). The main content area has a 'Close Community' button at the top. Below it, the 'Details' section shows: Contract 2773, Device Id 1741A4B4D5214BA495E9B92E3A944A3A, and Kit Colour None. A 'Select a kit colour' section features a grid of color swatches: None (selected with a green checkmark), Camo, Black, Grey, White, Red, Orange, Yellow, Lime, Green, Cyan, Blue, Navy, Purple, and Pink. Below the grid is an 'Enter device serial number' text input field. A note reads: 'Find the serial number for your iPad Pro or iPad on the back of the device.' At the bottom, there is a 'License Current Device' button and a 'Feedback' link.

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The screenshot shows the iSimulate web interface. The top navigation bar includes 'Home', 'Scenarios', 'Licensing', 'TMS', 'Device', 'Notifications', and 'Me'. The left sidebar contains sections for 'My Profile', 'My Content', 'Provisioning', 'Pre-Release', 'Resources', and 'Support'. The main content area is titled 'Provision Device' and features a 'Current License' section with a 'Revoke' button. The license details are as follows:

Details	
Agreement ID	2478
Device ID	A75995E0071B4351B034C2076B067D2F
Device Serial	DMPFH8P1PV03
Current Kit Colour	Black

Modules Included: CTGi Generic

Below the license details, there are sections for 'Your Standalone Licenses' and 'Your License Agreements'. A blue banner states 'Your device is currently licensed to this agreement'. Under 'Your License Agreements', it shows 'Ctgi License Agreement' with 0 licenses available. Details for this agreement are:

Details	
Type	Recurring
Licenses on agreement	1
Licenses used	1
Agreement ID	2478

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