


REALiTi SimScope – Set-up Guide


Wi-Fi Setup

1. Open REALiTi app on your iPad and choose Control mode
2. Click on "Stethoscope Tab" and tap on "Settings" (Gear icon), (Fig. 1).

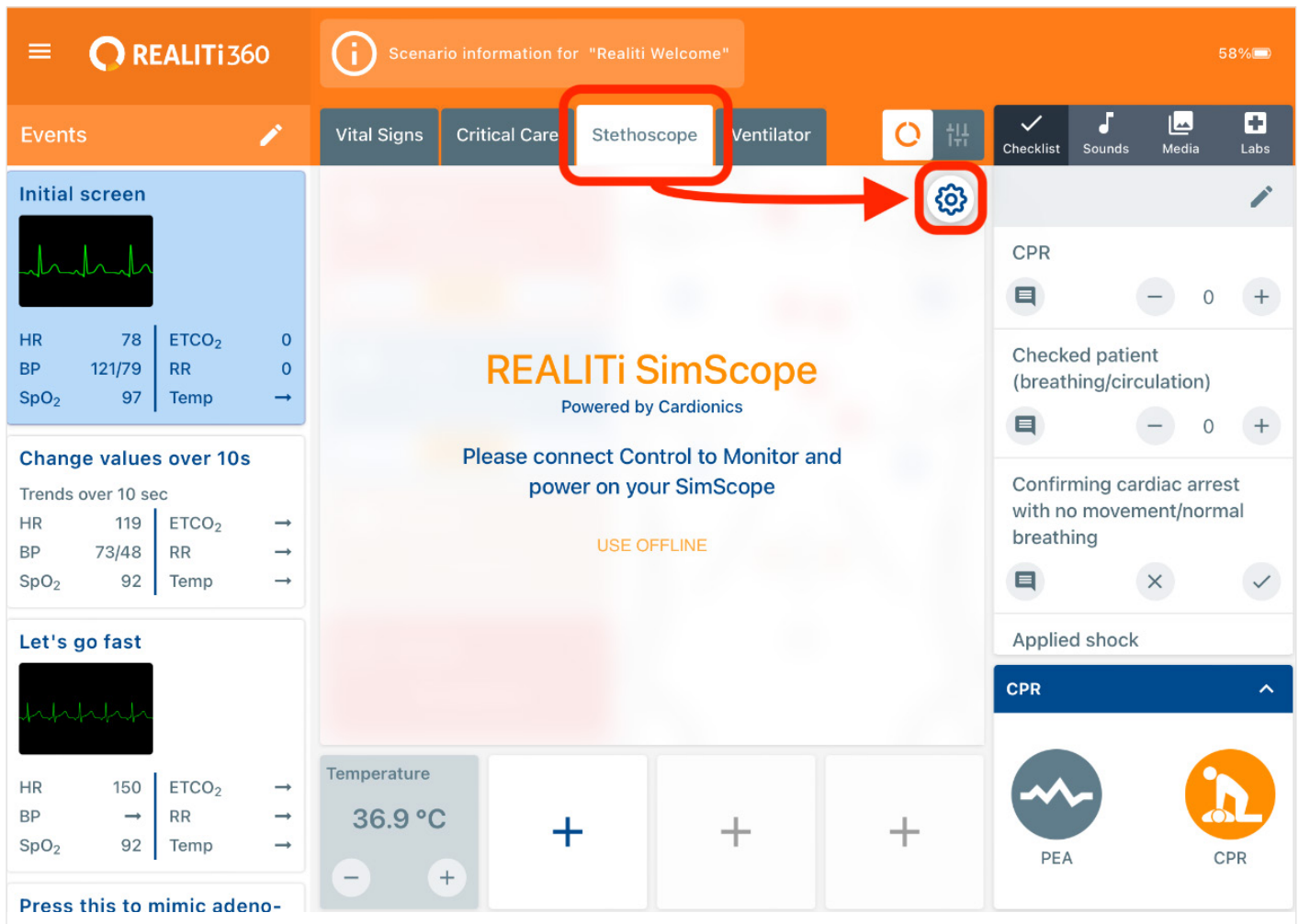

(Fig. 1)


3. You will see the Stethoscope Settings Screen (Fig. 2)

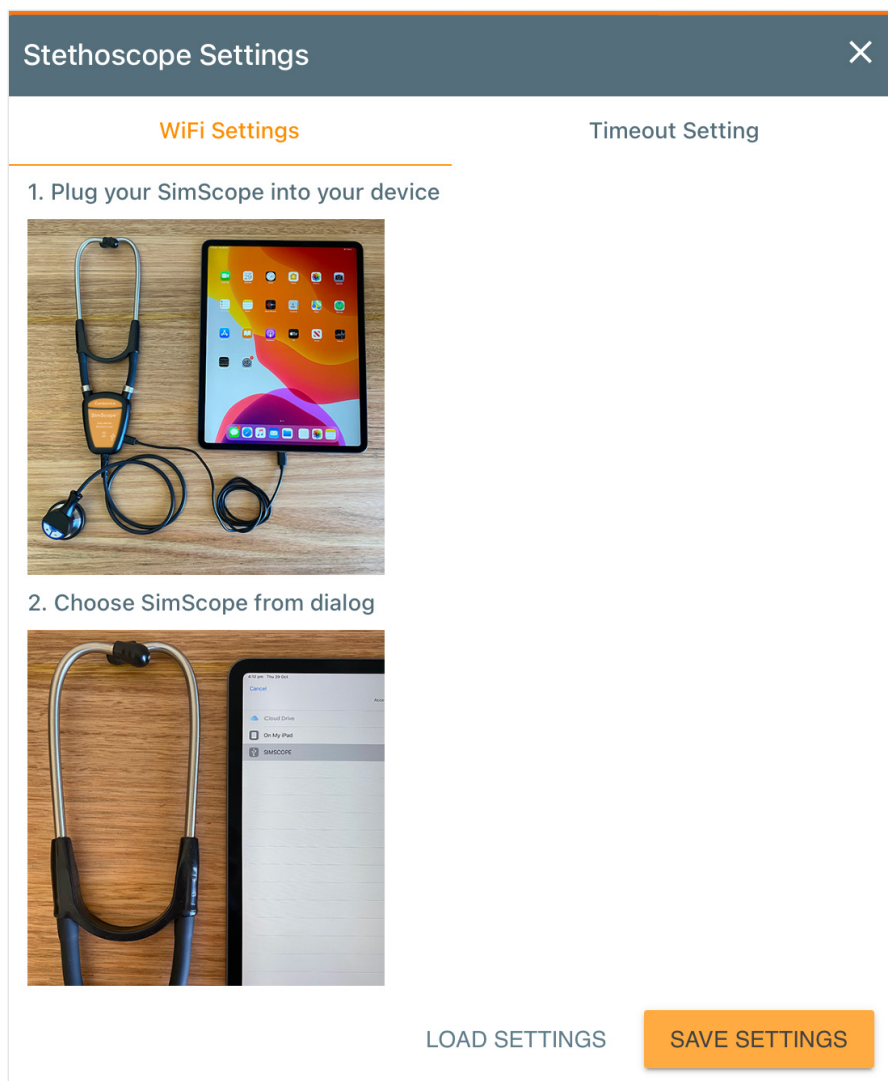

(Fig. 2)

Connecting REALiTi SimScope to your iPad and Configuring Wi-Fi

1. Plug the REALiTi SimScope into the iPad using the cable provided
2. Once the REALiTi SimScope is plugged in, tap on "Load Settings" (Fig. 2)
3. Wait for "SIMSCOPE" to appear on the file list, this might take some time. Then select "SIMSCOPE" and tap on "Done" (Fig. 3). Wait for the next screen to appear.

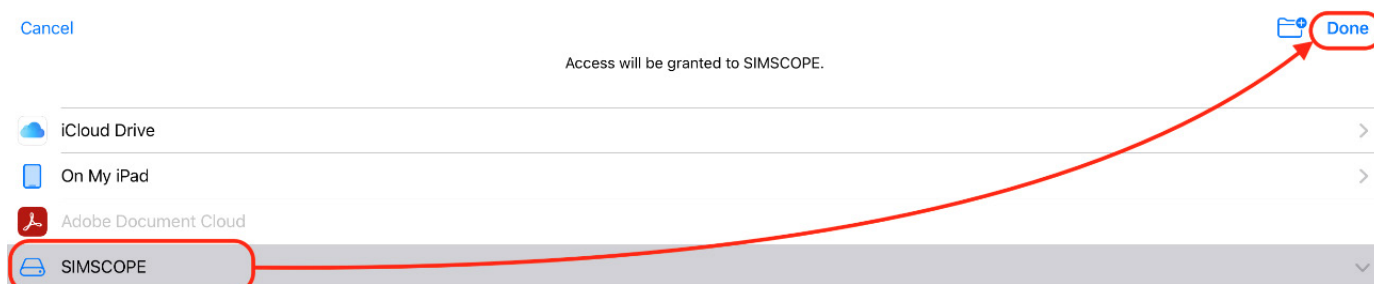

(Fig. 3)

4. You will then see options to add the Name and Password for your Wi-Fi. Once you have entered these, tap on “SAVE SETTINGS” (Fig. 4).

Note: Wi-Fi Name and Password should be same as the iPad is connected to.

5. Close the Stethoscope Settings box by pressing the “X” in the top right corner (Fig. 4).


The screenshot displays the 'Stethoscope Settings' interface. At the top, there is a dark blue header with the title 'Stethoscope Settings' and a close button 'X' in the top right corner. Below the header, the interface is divided into two main sections: 'WiFi Settings' (highlighted in orange) and 'Timeout Setting'. Under 'WiFi Settings', there are two tabs: 'Basic' and 'Advanced'. The 'Basic' tab is selected. It contains two input fields: 'WiFi Name' with the text 'iSimulate-Green' and 'Password' with the text 'XXXXXXXXXXXX'. Both input fields are circled in red. A red arrow originates from the right side of the 'Password' field and points down to the 'SAVE SETTINGS' button at the bottom right. At the bottom, there are two buttons: 'LOAD SETTINGS' (grey) and 'SAVE SETTINGS' (orange).

(Fig. 4)

NOTE: In order to set up a successful Wi-Fi connection, both the router as well as the REALiTi SimScope have to be switched on and active.


Testing the Connection

1. Ensure your chosen Wi-Fi router is on and working
2. Unplug the REALiTi SimScope from the cable
3. Place the earpiece of the stethoscope in your ears
4. Now press the plus (+) button provided on the bell part of the stethoscope.
5. You should hear the following:
 - 'System Activation'
 - 'Mode 1'
 - (A series of) 'beeps'
 - 'Wi-Fi Connected'
6. If this does not happen, plug the REALiTi SimScope back into the iPad and try again.
7. Now connect both the REALiTi iPads as usual (Fig. 5)

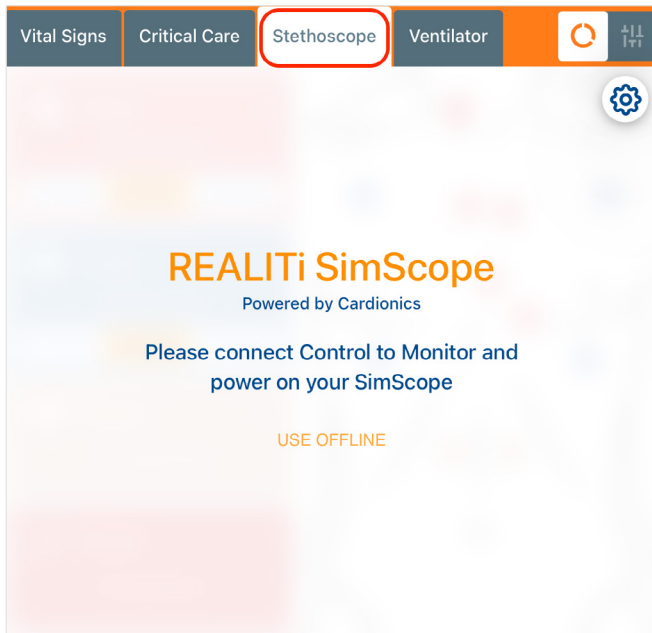


(Fig. 5)

8. Press the "Connect" button on the screen as shown below (Fig. 6)

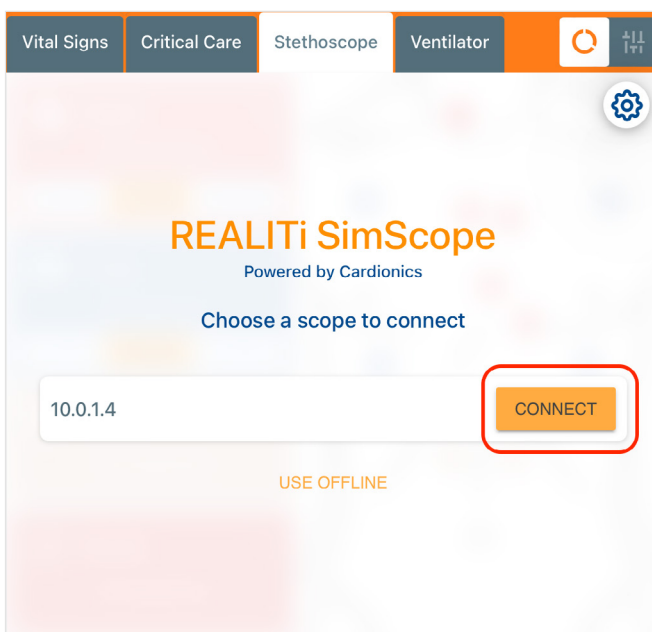


(Fig. 6)

The first time the REALTi SimScope connects, it will download the Sound Library (Fig. 7).

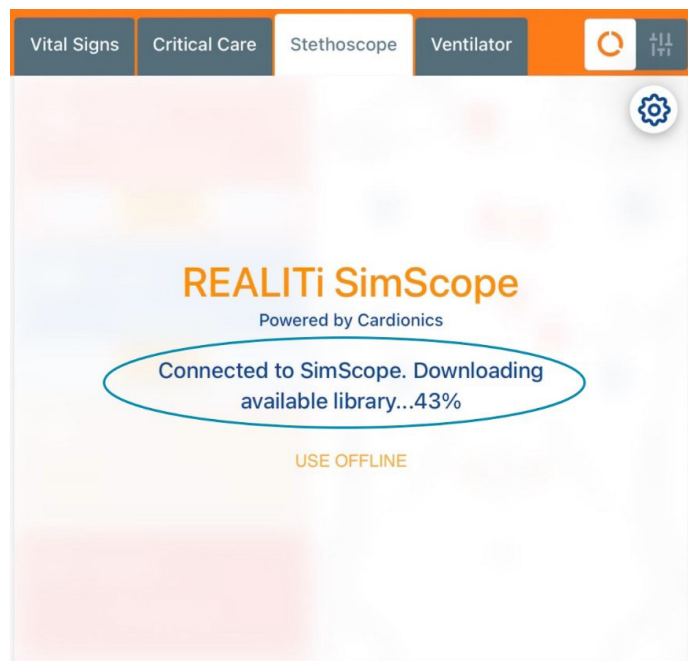

(Fig. 7)

9. Once the library has downloaded, you will see the main REALTi SimScope control panel (Fig. 8).

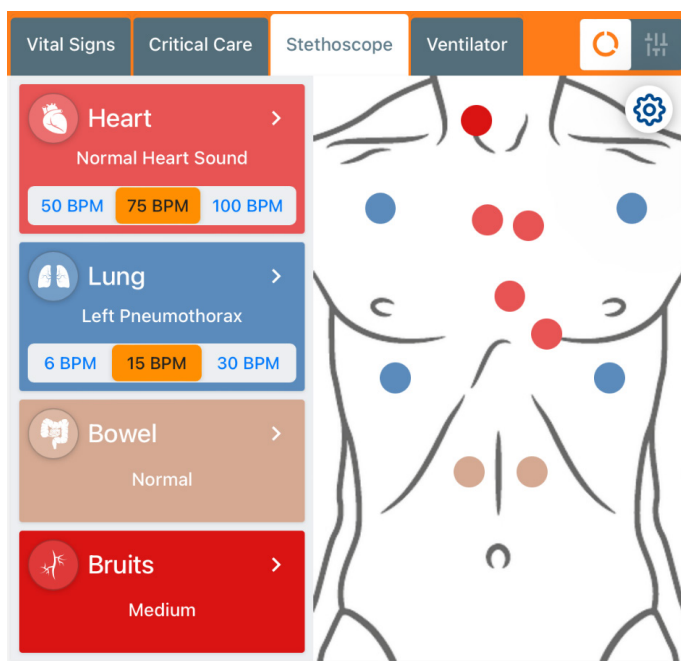

(Fig. 8)

REALTi SimScope is now ready to use!

Note: This is one time procedure only; you are not required to repeat these steps before each use.

➤ TROUBLESHOOTING TIPS

Followed all the steps above but the REALTi SimScope did not connect.

1. Ensure that both Control and Monitor iPad are on same Wi-Fi network and are connected. Ensure also that the Wi-Fi Name and Password on both of your iPads and in REALTi SimScope Settings are same.

Tip: It is recommended to use the Router that came with your REALTi 360 kit.

2. Set-up the REALTi SimScope again from the beginning.

3. The Stethoscope does not switch-on on pressing + button or switches-off soon. Replace the battery with a new one.

Tip: Use only AAA Lithium Battery. We recommend "Energizer Ultimate Lithium AAA" battery for optimal function.

If you are still not able to resolve the issue, please contact us.